

Canon



Alphenix 4DCT

Plan. Treat. Verify.



Integrate modalities with ease

Alphenix 4D CT perfectly integrates also with Canon Medical's ultrasound solutions with dedicated transducers, imaging and navigation tools to further enhance confidence and accuracy during interventional procedures and their follow-up.





Alphenix 4DCT




Quickly diagnose, treat and verify
with confidence.

Alphenix 4D CT seamlessly integrates our flexible Alphenix interventional system with the advanced Aquilion CT imaging suite into one versatile solution. With the ability to see, diagnose, plan, treat and verify in the same room, Alphenix 4D CT helps you prioritize safety, speed and efficiency during complex interventions.



Tightly integrated to help you boost productivity and patient care

Greater than the sum of the two parts, Canon Medical's Aquilion CT imaging suite and the Alphenix angiography system work in concert to help you make real improvements in efficiency, workflow and clinical confidence.

Conventional workflow



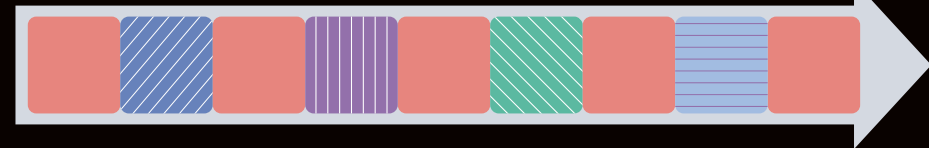


Alphenix 4D CT

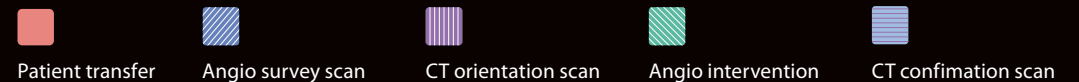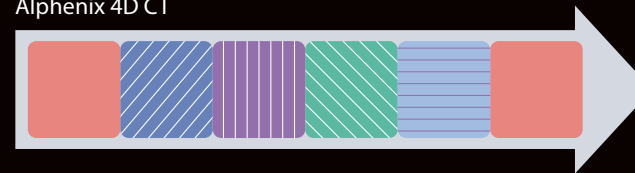

Eliminate patient transfer and improve efficiency

With the Alphenix 4D CT there is no need to transfer patients between rooms to confirm device placement or procedural success. Switching between CT and angio systems is smooth and fast. Both systems can also be used in tandem without the need to park one of them.

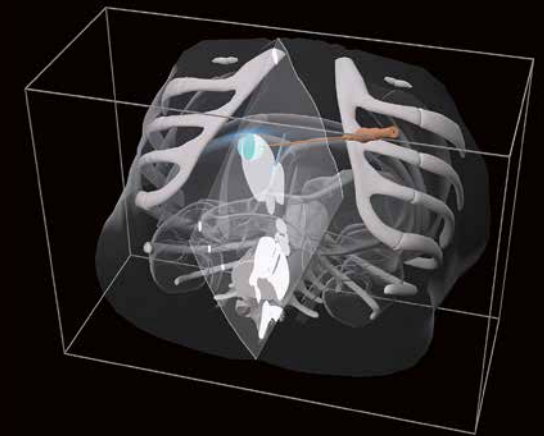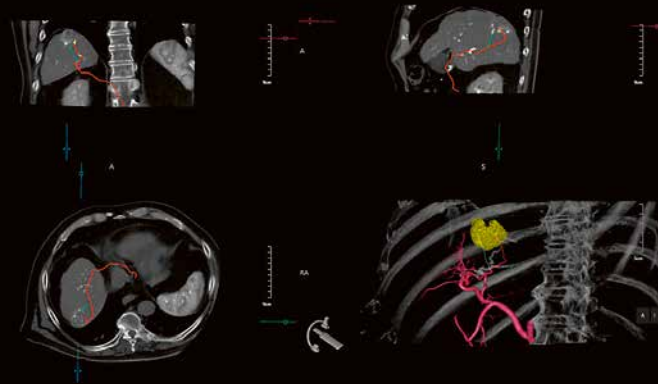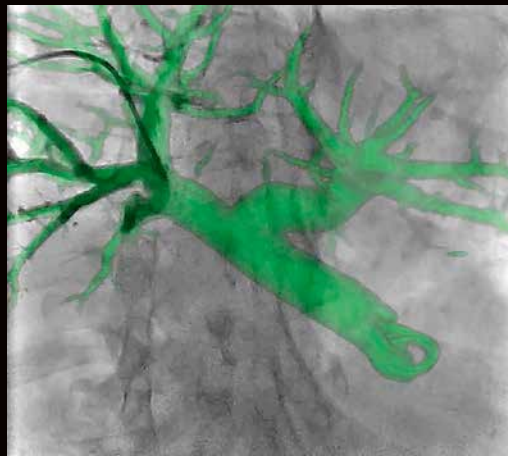

Navigate in 3D with confidence and accuracy

3D roadmapping enables you to superimpose a 3D-reconstructed vessel and device-enhanced mask volume on the fluoroscopic image for better visualization of the target anatomy and improved guide wire and catheter manipulation.

Embolization planning

The Embolization Plan software allows for automatic or manual tumor segmentation, safety margin definition around the segmented tumor and automatic bone segmentation for better viewing.

CT and angiography in sync

^{SURE}Guidance synchronizes the location of the region of interest between the two modalities. The CT gantry, angiographic C-arm and table move automatically, enabling you streamline workflow and prioritize procedural efficiency.

High resolution, low dose imaging right at your fingertip

Canon Medical's CT scanners employ a full host of advanced imaging technology to deliver high-quality images with an optimal balance of image quality and dose. Advanced post-processing tools help simplify complex protocols and provide consistent quality results.



Sharp, clear and distinct images at low dose

True CT imaging in the same room can deliver improved image quality and better low contrast detectability. Canon Medical's AI-driven Advanced Intelligent Clear-IQ Engine (AiCE) technology reconstructs images that are sharp, clear and distinct.*



Pixel perfect imaging, low iodine and patient dose

^{SURE}Subtraction Iodine Mapping helps you enhance your diagnostic capabilities with colored iodine maps provided automatically for any multi-phase abdominal protocol.


* AiCE is available on Aquilion ONE/GENESIS Edition and Aquilion Prime SP as option.





Genuine CT offers many advantages over cone beam CT. More flexible imaging with better image quality and faster acquisition, and last but not least, a significantly lower effective dose help you perform complex procedures efficiently and safely.

IMAGING	CONE BEAM CT	ANGIO CT
image quality	limited	optimal
FOV/coverage	limited, fixed	extended, flexible
contrast-enhanced imaging	single multiple phase	multiple phases
WORKFLOW		
preparation/iso centering	time-consuming	simple, fast
patient breathing	high impact	negligible
post-processing	complex	highly automated
DOSE		
effective dose	high	low



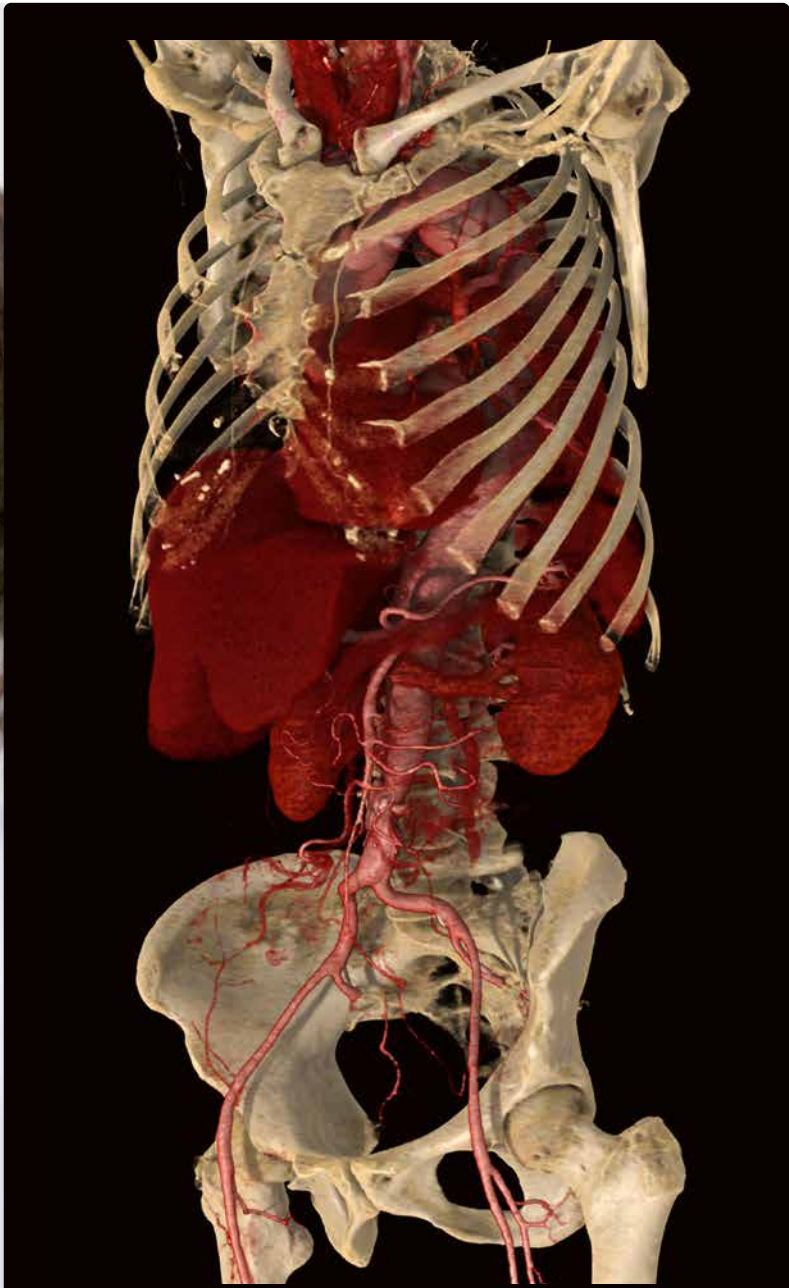


Better patient access, extended field of view

The extra-wide, flared gantry of our CT scanners provides outstanding patient access from front and rear. Compared to cone beam CT, true CT scanning offers the ability to acquire images with an extended field of view faster and at significantly lower dose.

Crystal-clear imaging, superior versatility

Alphenix 4D CT is powerful hybrid imaging system that combines the world's most flexible angio suite with the most advanced dynamic volume CT. Delivering an outstanding solution for image-guided interventions the system covers a wide range of interventional procedures.

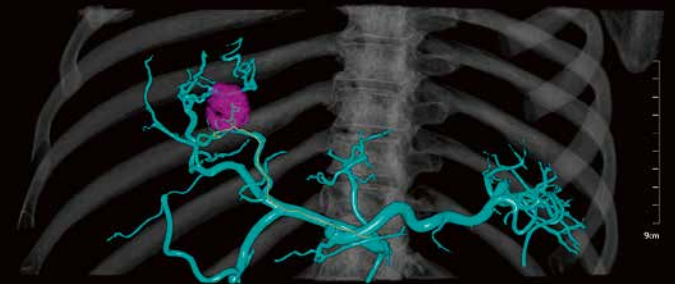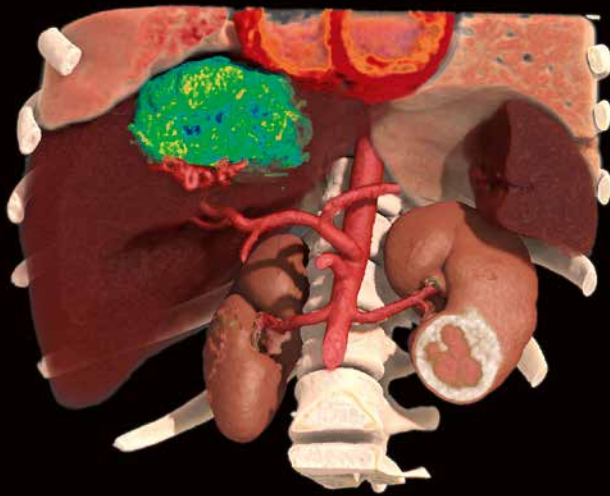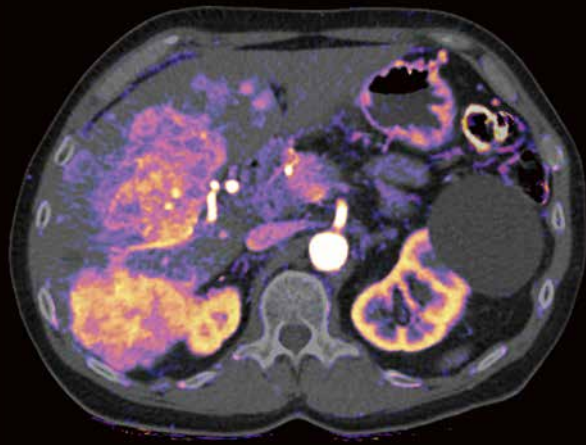

Advanced perfusion imaging assists you in imaging and treatment planning of tumors more accurately and immediately verifying the response to therapy. Alphenix 4D CT simplifies the positioning of micro-catheters and needles with 4D imaging of tumor vascularization and real-time 3D needle tracking.



Imaging morphology and function

Canon Medical's wide detector CT technology with up to 16 cm coverage in a single rotation allows you to acquire anatomical and functional data simultaneously with high spatial and temporal resolution.

Diagnose and treat with speed and flexibility

Speed is crucial. Robust diagnostic images can be captured in a fraction of a second, enabling you to minimize dose exposure, image artifacts and procedure time. With the flexibility of the interventional suite, you can treat without moving the patient. Across a wide range of clinical applications, from oncology, cardiology, neurology and trauma care, Alphenix 4D CT offers possibilities to deliver safe and efficient treatment to your patients.

Alphenix Sky

- 30x30 cm or 30x40 cm FPD
- FPD & collimator sync
- 270° ceiling rotation
- Lateral C-arm stroke



Standard table

Alphenix Sky+

- 30x40 cm FPD
- FPD & collimator sync
- 270° ceiling rotation
- Lateral C-arm stroke
- 3D full body coverage



Tilt/cradle table

Aquilion

- 90 cm bore
- 32 slices



Aquilion Prime SP

- 78 cm bore
- 80 slices (upgradable to 160 slices)
- AiCE



Aquilion ONE GENESIS Edition


- 78 cm bore
- 640 slices
- 16 cm coverage / rotation
- AiCE



Configure Alphenix 4D CT with the CT that meets your needs

Alphenix 4D CT is available in a wide range of configurations to provide an optimal solution for diverse clinical environments.

Cardiac intervention

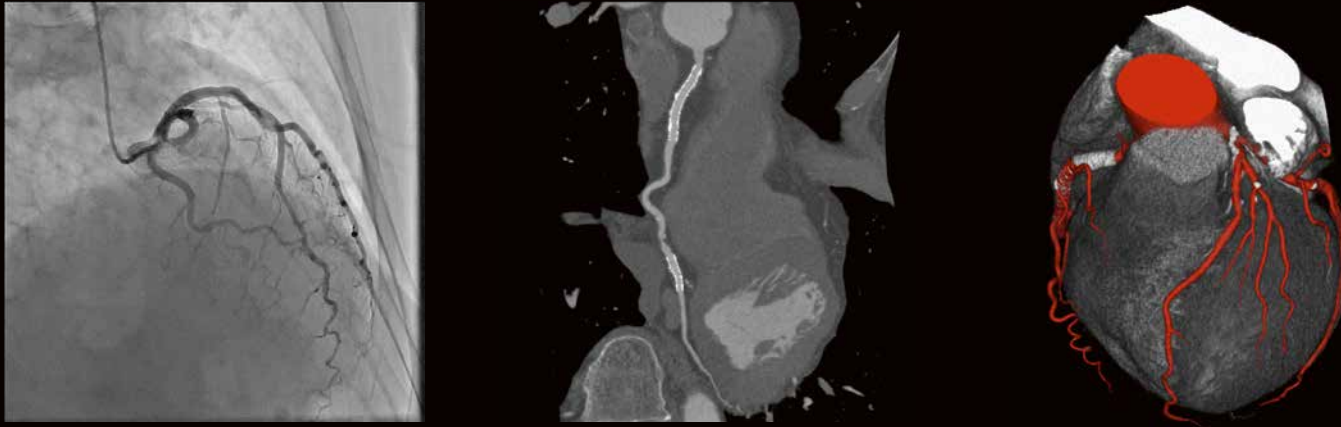

Alphenix 4D CT provides robust low-dose cardiac CTA. One rotation is all it takes to capture the entire heart in just a split second with the Aquilion ONE. Eliminating motion artifacts, this technology is perfect for planning interventional procedures and for verifying percutaneous stent or valve placements and repairs immediately at bedside.

Neuro/stroke



Canon Medical's area detector technology delivers iso-phasic 4D vascular imaging without table or gantry movement for robust evaluation of fistula and aneurysm treatment. The Neuro ONE stroke protocol provides dynamic perfusion mapping in under a minute for fast and reliable diagnostics in acute stroke patients.


Trauma



Speed is foremost important in managing trauma patients. With Alphenix 4D CT you are now capable of diagnosing and treating without the need to transfer the patient between rooms, saving valuable time.


One room. One system. Many procedures.

The Alphenix 4D CT system combines the interventional lab with a complete CT imaging suite for a more streamlined workflow and a smoother, safer patient experience in a single clinical setting. You no longer need to transfer the patients between departments – Alphenix 4D CT enables you to plan, treat and verify in a single setting for better patient care and higher productivity.

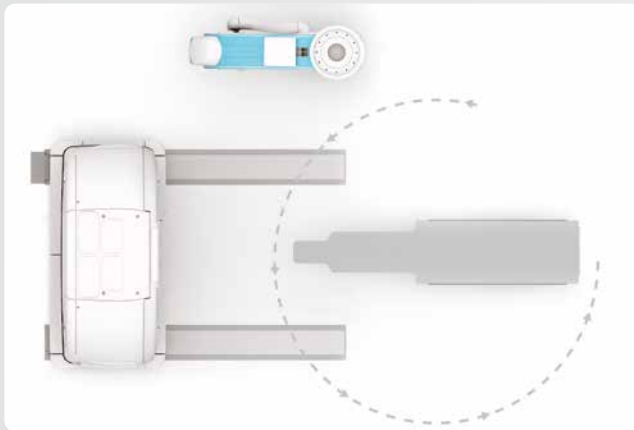


Free patient access whenever needed

Convenient parking positions for both the C-arm and the CT gantry ensure free access to the table for patient mount and emergency procedures.

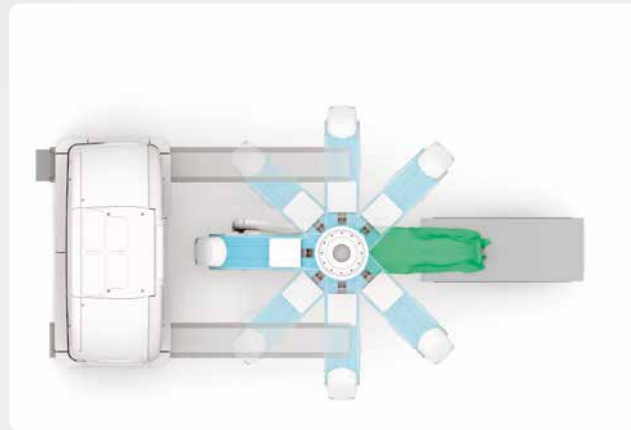

Freedom to move with your workflow


Alphenix offers a 270° range of movement allowing you to position the C-arm to suit your workflow even during the most complex interventional procedures.

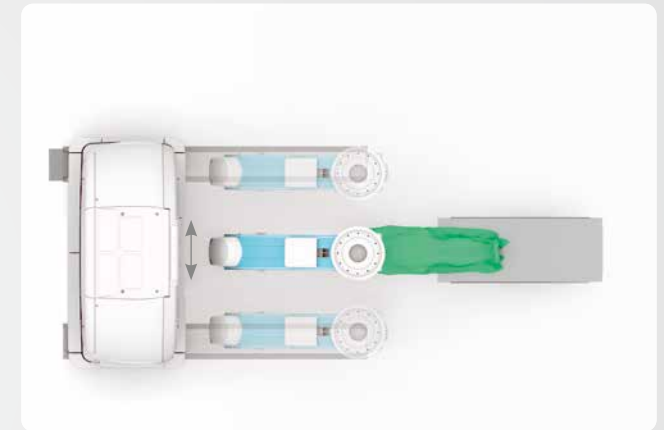

Unique lateral coverage to aid transradial procedures


The dual-track ceiling-mounted C-arm of the Alphenix 4D CT provides unique fingertip-to-fingertip coverage during transradial and transbrachial procedures.



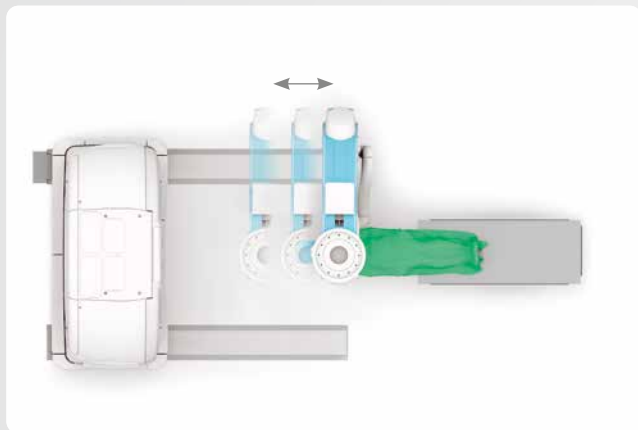


Smooth workflow with head-to-toe access

While the CT gantry can be fully removed from the workplace, Alphenix' five-axis technology provides faster and better positioning of the C-arm from head to toe.

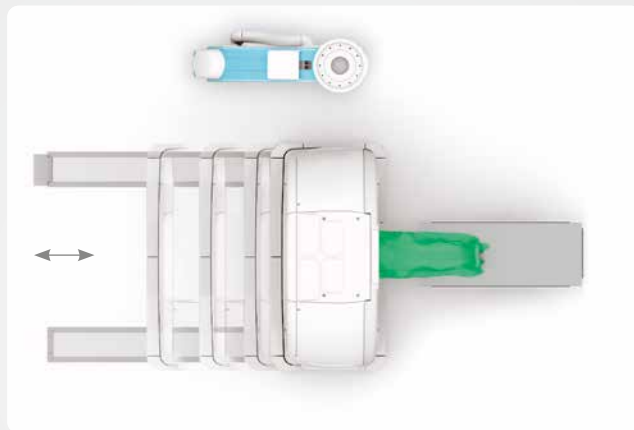

Fine detail visualization

Advanced CT imaging with the industry's only 0.5 mm detector element means greater resources for planning and verifying in the interventional space.

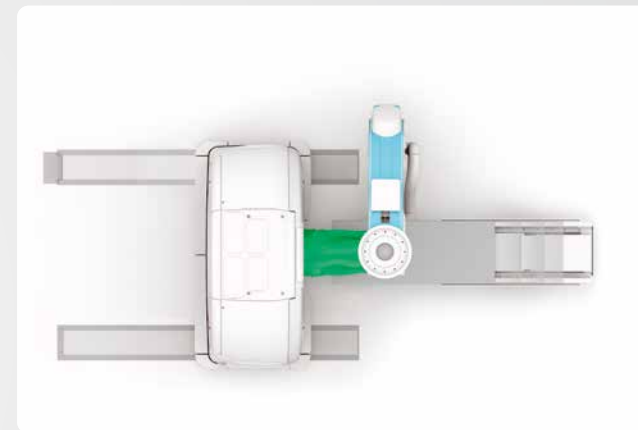


Seamless transition between modalities

Alphenix 4D CT markedly improves workflow with its ^{SURE}Guidance technology that manages seamless transition between modalities.

Better protection For patients and clinicians


Dose management is an important issue for everyone. Canon Medical has optimized every step of the imaging chain to reduce radiation dose while improving workflow and image quality. DoseRite is our comprehensive dose management solution. Combining advanced detector design with an array of powerful software features, it provides you with an extensive host of dose-saving and dose management functions.

Active targeted fluoro area
Surrounding reference area, no radiation

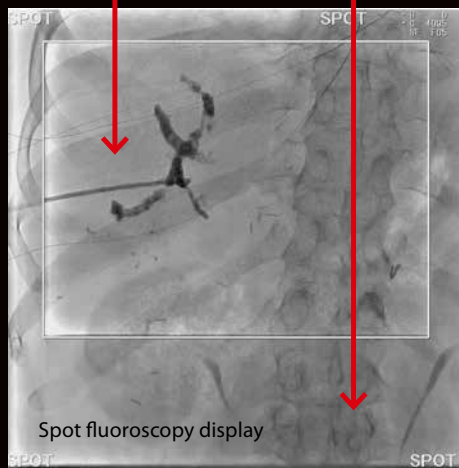


Spot Fluoro Display

Using Spot Fluoro can result in substantial dose reductions by displaying the full-frame Last Image Hold (LIH) around the live fluoro image, reducing the need to open up collimation for viewing landmarks outside the spot field.

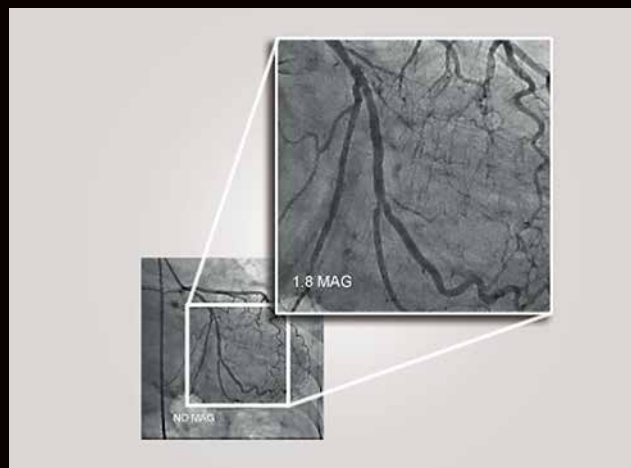

Live Zoom

Live digital zoom increases image display size in real time during both fluoroscopy and digital acquisition offering potential dose savings compared to conventional FOV magnifications.

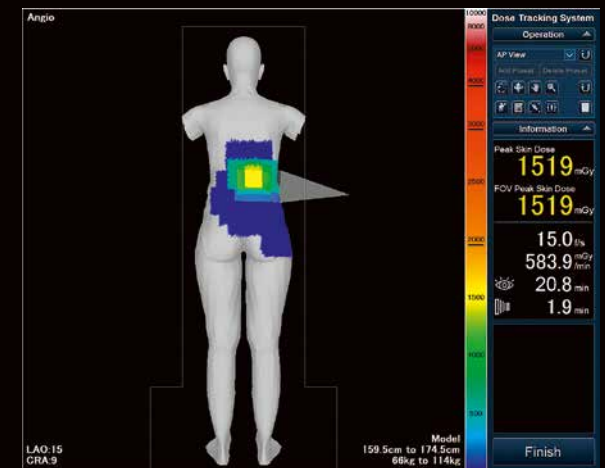

Dose Tracking System (DTS)

The operation screen of the Dose Tracking System, illustrating the color-coded skin dose distribution display area and the dose information display area that includes numerical peak skin dose and FOV peak skin dose estimates.



DoseRite

Alphenix 4DCT

Canon

CANON MEDICAL SYSTEMS EUROPE B.V.

<https://eu.medical.canon>

©Canon Medical Systems Corporation 2020. All rights reserved.
Design and specifications are subjected to change without notice.
Model number: INFx-8000C, TSX-201A, TSX-303B, TSX-305A
MCAVL0005EC 2020-03 CMSE/Printed in Europe

Canon Medical Systems Corporation meets internationally recognized standards for Quality Management System ISO 9001, ISO 13485.
Canon Medical Systems Corporation meets the Environmental Management System standard ISO 14001.

Alphenix, Aquilion, Aquilion ONE, Aquilion ONE GENESIS, ^{SURE}Subtraction and Made for Life are trademarks of Canon Medical Systems Corporation.

Disclaimer: Some features presented in this brochure may not be commercially available on all systems shown or may require the purchase of additional options. Please contact your local Canon Medical Systems representative for details.

Made For life